




REMEDIC

EYE DROP DISPENSER

Application aid for easy dispensing of eye drops

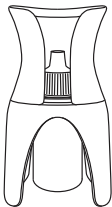

GB


- This application aid is suitable for most common eye dropper bottles.
- This eye drop dispenser reduces the risk of injury by preventing direct eye contact with the tip of the bottle.
- Eye infections caused by a dirty tip are thus prevented.
- The soft edge of the applicator holds the eye open and the drops land safely in the eye.
- Please read these instructions carefully before use.

Notes on eye drops


- It is recommended to warm the eye drops to approximately body temperature.

- Eye drops should only be used after consulting your doctor.
- Pay attention to personal hygiene when using eye drops.
- The exact handling depends on the product. In some cases there may be a need to remove contact lenses as they may cause interactions, especially if you use other eye drops. Please always read the package insert for the corresponding preparation.
- Before dispensing the drops in the eye, wash your hands thoroughly and dry them.
- Clean the eye drop dispenser regularly under running water and, if necessary, use some detergent.
- Allow the eye drop dispenser to air dry.
- Never clean the applicator with alcohol, acetone, petrol, aggressive detergents, etc.

Inserting the eye dropper bottle into the applicator

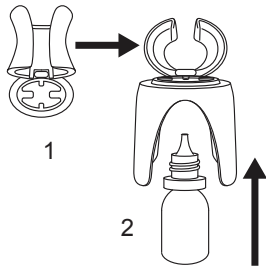


1. Flip open the applicator.
2. Remove the lid from the eye dropper bottle and insert the bottle into the applicator from below. Make sure the eye dropper bottle sits firmly in the applicator.

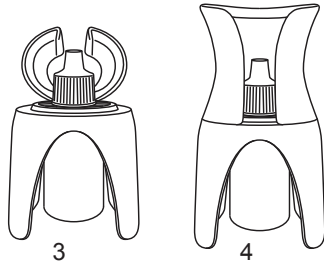


3. Close the eye dropper bottle.
4. Close the applicator with a snap.

Dispensing the eye drops

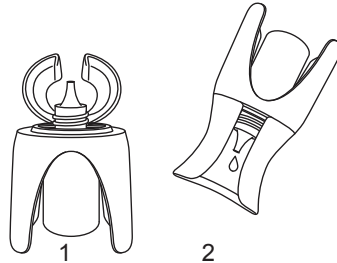

1. Flip open the applicator and remove the lid from the eye dropper bottle.
2. Close the applicator with a snap. The eye drop dispenser is now ready for use.

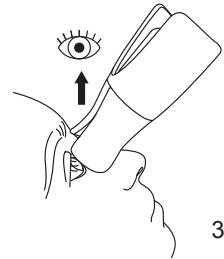

3. Position the eye drop dispenser on the eye and bend your head backward as far as you can. Look up and try not to blink while dispensing the eye drops. If necessary, slightly squeeze the eye dropper bottle.