REMEDIC

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REMEDIC

Water glass with pill box

The perfect combination for easy administration of supplements and other pills

Fill the water glass.

Load the pill box and put it on the glass.

At the time of taking pills lift the pill box from the glass and open it.

- The water glass made of borosilicate has a recess for the thumb for a secure grip and safe handling.
- The scale on the glass make it easier to monitor the liquid intake.
- The pill box is easy to open and thanks to a silicone ring it can be securely attached to the Glass.
- The flattened lid can easily be marked with a code or name tag and a transparent viewing window allows a look into the pill box.

Cleaning and care

- Before using for the first time, you should wash the water glass with pill box with hot Rinse off water.
- Do not use alcohol, acetone, Petrol or aggressive cleaning supplies.
- You can wash the glass in a dishwasher.
- The pill box is not dishwasher safe and is rinsed off with warm water. If necessary, use a mild detergent.
- Leave the glass and pill box open to dry.

Disposal

- Please use the return and collection systems in your community or contact the dealer from whom the product was purchased.

Produced for: