



REMEDIC

Door stopper with telescopic handle

The door stopper with 90cm telescopic handle holds doors effortlessly without that you have to bend down and up.

It prevents accidental slamming of doors and can be used for safety can also be placed from the inside.



1. Open the stopper and pull the telescopic handle to the desired length
2. Push the wedge under the door.
3. The stopper stays upright and can be effortlessly removed without bending down.

Cleaning and care

- Do not use alcohol, acetone, gasoline or aggressive cleaning agents
- If necessary, wipe the door stopper with a damp cloth.

Disposal

- Please use the return and collection systems in your Community or contact the retailer where the product is was bought.

For additional help/info
email: shop@remedic.co