


REMEDIC

SHOWER STOOL / DUSCH-HOCKER

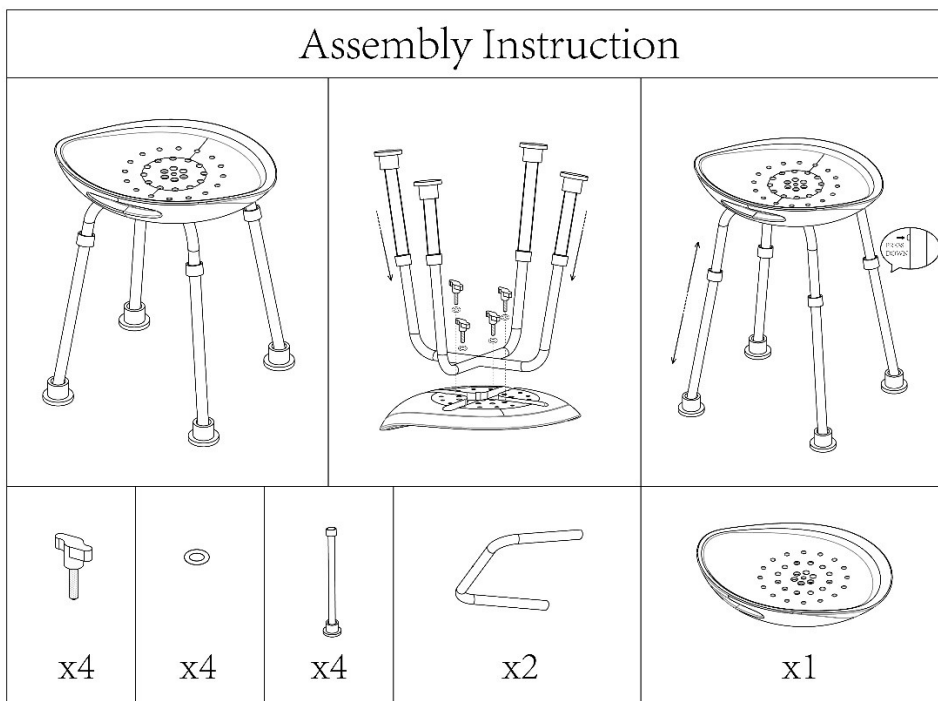



MAXIMUM USER WEIGHT 150KG / 300LBS

USE IT SAFE – USE IT ON EVEN NON-SLIPPERY SURFACES ONLY

BEFORE FIRST USE

BEFORE USING OR PLACING THE SHOWER STOOL PLEASE MAKE SURE THE SAME IS COMPLETE AND UNDAMAGED. MAKE SURE ALL RETAINING CLIPS ARE PROPERLY LATCHED IN PLACE. DO NOT USE THE SHOWER STOOL IF ANY ONE RETAINING CLIP DOES NOT PROPERLY LATCH IN PLACE.

INSTRUCTIONS FOR ASSEMBLY

- PUT THE SEATING SURFACE ON A PLANE SURFACE UPSIDE DOWN.
- PLACE THE FRAME PARTS ONE ON TOP OF THE OTHER CENTRALLY ARRANGED SUCH THAT THE RECESSES ENGAGE, AND FIX BOTH COMPONENT PARTS IN PLACE USING THE SCREW IN THE MIDDLE.
- NOW, FIX THE FRAME PARTS IN PLACE BY SCREWING DOWN THE REMAINING FOUR SCREWS.
- NOW, INSERT THE FOUR LEGS SUCH THAT THE BORES OF ALL LEGS FACE THE INNER SURFACE OF THE STOOL, AND MAKE SURE THE SPRING-LOADED PINS PROPERLY LATCH IN PLACE. PLEASE ADJUST ALL LEGS TO THE SAME LENGTH.

CARE AND SERVICING

THIS SHOWER STOOL DOES NOT REQUIRE ANY SPECIFIC SERVICING. IT MAY BE CLEANED USING ANY OFF-THE-SHELF CLEANING AGENT.

DISINFECTION

THIS SHOWER STOOL CAN BE DISINFECTED USING ANY COMMERCIALY AVAILABLE DISINFECTANT APPROPRIATE FOR USE WITH PLASTIC MATERIALS.

IMPORTANT

USE THE INSTRUCTIONS PROVIDED BY THE MANUFACTURER WHENEVER USING DISINFECTANTS.

NOTICE

CHANGES OR REPAIRS ON THE SHOWER STOOL MAY EXCLUSIVELY BE PERFORMED BY DULY QUALIFIED TECHNICAL PERSONNEL AND USING ORIGINAL SPARE PARTS ONLY.

SUBJECT TO CHANGE WITHOUT PRIOR NOTICE.