



# REMEDIC

Remedic Quick-Fixing Body Wrap Bandage – standard width

Size: Approximately 55 inches long and 6 inches wide

---

## Product Description

The **Remedic Quick-Fixing Body Wrap – Self Adhesive Band** is a reusable, self-adhesive wrapping band designed for versatile applications. Made from 100% polyester, it offers a soft, breathable, and comfortable experience while providing warmth and protection for sensitive body areas. This sustainable wrap is ideal for securing heat or cold therapy packs, wrapping sensitive areas, or providing gentle compression

---

## Key Features

- **Material:** 100% polyester for comfort and durability.
  - **Reusable:** Machine washable at up to 140°F (60°C) for long-term use.
  - **Self-Adhesive:** Lighter-colored underside adheres to the darker top side, with adhesive strength increasing with the size of overlapping area.
  - **Breathable:** Keeps you warm while allowing airflow, preventing discomfort.
  - **Versatile:** Can be used in heat/cold therapy, compression, or wrapping sensitive areas.
  - **Noiseless and Gentle:** Easily applied and removed without noise or force, suitable for sensitive skin.
- 

## Applications

1. **In Heat Therapy:**
  - Secure a hot water bottle to stay in place, even when you move around.
  - Use as a warming wrap for comfort (e.g. use it as a scarf that stays in place)
2. **In Cold Therapy:**
  - Attach an ice pack for targeted relief.
3. **Compression:**
  - Use as a compression wrap.
4. **Skin Protection:**
  - Cover sensitive skin after applying ointments.

- Gently wrap sensitive body parts when required.
- .... and many more

---

## How to Use

1. **Prepare:** Position the bandage on the area you wish to wrap (e.g., abdomen, back, or upper body).
2. **Apply:** Unfold the bandage and wrap it around the area snugly, aligning the lighter underside to stick to the darker top side.
3. **Secure:** Overlap the material to increase adhesive strength. Ensure it is firm but not restrictive.
4. **Adjust:** Modify the tightness for a comfortable fit that allows freedom of movement.

**Note:** The bandage can also be used to hold heat or cold therapy packs securely in place.

---

## Care Instructions

- **Washing:** Machine washable cold or hot up to 140°F (60°C)
- **Drying:** Air dry or tumble dry on low heat.
- **Avoid:**
  - Bleaching: Do not use bleach as it may damage the material.
  - Ironing: No ironing required.

---

For further assistance or information, please contact us at:

**Email:** [shop@remedic.co](mailto:shop@remedic.co)

**Website:** [www.remedic.co](http://www.remedic.co)

---

This wrap designed with sustainability in mind and can be reused multiple times to provide comfort, protection, and relief wherever needed. The ability to cut the bandages ensures a custom fit for every user.

Article No.	REM280119
GTIN - EAN No.	5060859280119

Made in China