

USER MANUAL

*Please read this note carefully, the actual situation shall prevail.

Product Features

1.Product Specifications

Electric Grinding, Time-Saving:
900mAh large-capacity 18350 lithium battery +
22500RPM high-speed motor for quick and efficient
grinding.

2.Uniform Grinding, Faster Cutting:

Four-blade 40° bending design ensures even cutting
of herbs/tablets.

3.Easy Operation:

Press lightly to grind; automatic stop after 20
seconds.

4.Safe Storage and Use:

Dual-lock lid design prevents spills; built-in safety
switch for added protection.

5.Clear Charging Indicators:

Red light during charging, green when fully charged.
Low battery reminder (red light).


6.Easy Cleaning:

Detachable material bin (dishwasher-safe); wipe the
main unit with a damp cloth.

7.Portable Design:

Compact and travel-friendly with safety locks.


Product Description

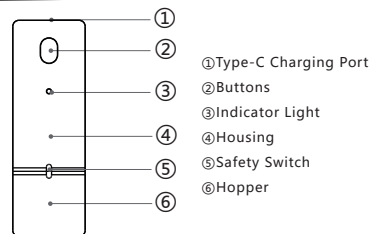


Product Specifications:

Product Number	1050083
Full Charge Time	2.5hours
Input	5V-1A
Product Dimensions	Φ42.6*125.5mm
Product Weight	202g
Hopper Capacity	20mL
Power	16.8W
Rated Voltage	3.7V
Battery Capacity	900mah
Material	Hopper: ABS Main Unit Components: SS304 Grinding Blades: SS304 Buttons: ABS


Operating Instructions

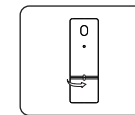


1. Turn hopper counterclockwise to open.

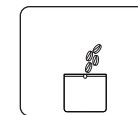

2. Add grinding materials.

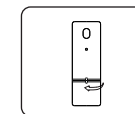


3. Turn hopper clockwise to close.

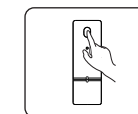


4. Press switch 3x* (hold & shake).



5. Hold & shake

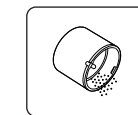

6. Remove powder/particles.

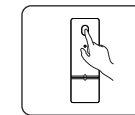

7. Press switch 3x to off (or auto-shutdown in 8min).

Press the switch three times to turn on. The indicator light will turn blue.
If the red light is on, charge the device or check if the hopper is properly closed.

Caveat

To ensure safety and maintain the high quality of
this product, we recommend that you read the
instruction manual carefully and follow these
instructions to the letter.

Initial Charging: Fully charge before first use. Do
not operate while charging.

Cleaning:

Main Unit: Never immerse in water. Use a brush or
dry cloth.

Material Bin: Wash with water; dry thoroughly.


Do Not Separate During Use: Bin and main unit
must remain connected.

No User Repairs: Contact authorized service
centers to avoid warranty voidance.

Keep Away from Children.

Usage Environment: Avoid high temperatures,
flammable materials, or explosive gases.

Charging Precautions

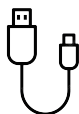

Charging Instructions:
Charge when the red light flashes
during grinding. The red light
indicates charging, and it turns green
when fully charged.

- 1.This product uses a lithium battery. For optimal
battery life, recharge periodically when not in
use.
- 2.Use a power adapter with an output voltage of
4.5V-5.5V for charging.
- 3.The built-in battery is non-removable. Do not
attempt to replace it yourself to avoid damage.
- 4.Avoid exposing the battery to high tempera-
tures or heat sources (e.g., direct sunlight,
heaters, microwaves, ovens). Overheating may
cause explosions.
- 5.Do not disassemble, modify, insert foreign
objects, or immerse in water/liquids to prevent
leaks, overheating, fire, or explosions.
- 6.If battery fluid leaks, avoid skin/eye contact.
Rinse immediately with water and seek medical
help if exposed.
- 7.Do not incinerate the battery to prevent fire or
explosions.
- 8.Dispose of the battery per local regulations. Do
not discard as household waste. Improper
disposal may cause explosions.

Indicator Status

Red Light	Battery <30%
	Safety Switch Open
	Motor Blocked(Blinks 3 times)
Blue Light	Charging
	Battery ≥30%
	Safety Switch Closed
Green Light	Normal Grinding
	Overload(Blinks 3 times)
Green Light	Fully Charged



REMEDIC

<http://www.remedic.co>
shop@remedic.co

Made in china

FAQ

- Q: How to clean the grinder?
A: Wipe the exterior with a damp cloth. Use
a brush for residue. Never wash the main
unit with water.
- Q: Why does grinding pause every 20
seconds?
A: Designed to prevent overheating (20
seconds grinding, 20 seconds pause).
- Q: What materials can be ground?
A: Brittle items like tablets, salt, or dry
herbs.